

Call Absolute Hearing Solutions
614-452-4280

life-changing
technology

OTICON | Ruby

Made to move

Sound quality, connectivity and rechargeability
in one complete essential solution



oticon
life-changing technology

Hearing moves us in many ways

Oticon Ruby – setting a new standard for the essential hearing aid category

The ability to hear connects us to the world in so many ways. When your patients are engaging in a wonderful conversation with friends, connecting via the phone or just listening to their favorite song, the last thing they want to be worried about is their hearing. Life moves fast and today's patients need hearing devices that can keep up.


Introducing Oticon Ruby, our new essential line of behind-the-ear hearing aids designed to give your patients the confidence they need to live life to the fullest.

Oticon Ruby provides the best Oticon sound quality ever offered in the essential category. It also meets the everyday needs of your patients with outstanding wireless connectivity and a lithium-ion rechargeable solution.

Oticon Ruby joins our existing Oticon Siya custom range and is available in a CROS/BiCROS solution to offer your patients the styles and options they need. Together Oticon Ruby and Oticon Siya combine to complete our most comprehensive essential product portfolio ever.



Apple, the Apple logo, iPhone, iPad, and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc. Android, Google Play, and the Google Play logo are trademarks of Google LLC.

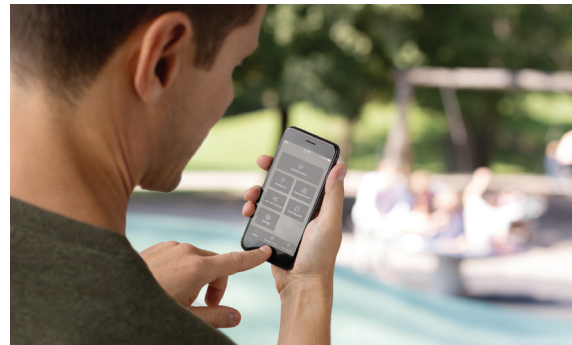

Made to capture special moments

Oticon Ruby's best in class sound quality is powered by our most advanced platform ever - Velox S™. Ruby combines a range of innovative BrainHearing™ technologies including: Noise Reduction LX, Multiband Adaptive Directionality LX, Single Compression LX and more, to deliver a superb sound quality your patients desire.

Oticon Ruby includes our **new SuperShield feedback system** designed to stop annoying whistling before it occurs. Gone are the days when your patients lived in fear of experiencing embarrassing whistling emanating from their hearing aids when hugging a loved one, talking on the phone or even just sitting in the car.

SuperShield works together with Feedback shield LX to analyze the incoming sound levels, identify feedback risk and quickly and accurately prevent whistling. This gives your patients not only the amplification they need but also peace of mind as they move throughout their day.

Oticon is constantly challenging conventions. That's why our whole approach to audiology is about much more than just compensating for damage to the ear. We design our hearing aids with the philosophy that better hearing starts with the brain, so we constantly strive to deliver what the brain needs to make sense of sound. We call this BrainHearing.



SuperShield

Rapidly and intelligently prevents feedback before it occurs



Feedback shield LX

Maintains effective feedback suppression over time



Multiband Adaptive Directionality LX

Quickly adapts to changing soundscapes by fluently applying directionality



Single Compression LX

Compresses and amplifies sounds into the audible range



Noise Reduction LX

Attenuates disturbing noise extremely quickly, even in-between words



Speech Rescue™ LX

Makes high frequency sounds more audible



Wind Noise Management

Improves access to speech in situations with wind noise



Tinnitus SoundSupport™

Relief sounds for patients with tinnitus

Made to connect

Oticon Ruby allows your patients to take advantage of today's connected world. Ruby includes 2.4 GHz Bluetooth® low energy technology for wireless connectivity and high-quality streaming to a wide range of devices such as **smartphones, TVs, computers** and more.

Oticon Ruby accessories and apps



ConnectClip

Use ConnectClip as a remote microphone, remote control or to turn your patients hearing aids into a wireless stereo headset



TV Adapter

Oticon's TV Adapter streams the sound from a TV directly to your patient's hearing aids



Remote control

Allows your patients to adjust volume, switch programs or turn the hearing aids off with the touch of a button



EduMic

Connects your pediatric patients' hearing aids directly to teachers, microphone systems, computers, tablets and more in the classroom



Oticon ON App

Switch between programs, adjust volume, use HearingFitness™, Internet of Things and much more with Oticon ON App



Made for a full day's power.* Every day.

Oticon Ruby brings a state-of-the-art lithium-ion rechargeable solution to your essential patients. Ruby gives your patients a reliable rechargeable solution they can count on to help them move worry free throughout their day.

Oticon Ruby miniRITE R provides:

- Super fast three-hour charging time for a full day of power, including streaming
- Quick recharge - 30 minutes delivers an additional six hours of power
- Contact-free and reliable, inductive charging
- Intuitive-to-use, stable and stylish charger



*Lithium-ion performance varies depending on hearing loss, lifestyle and streaming behavior

Made to meet your patients' needs


OTICON | Ruby

Oticon Ruby behind-the-ear instruments are powered by our newest platform, Velox S. Offering Oticon sound quality, wireless connectivity and rechargeability, Oticon Ruby is Oticon's strongest essential product ever offered. Ruby includes a choice of performance levels, styles and fitting ranges to match your patients hearing loss, design preferences and communication needs.

| Benefits | Features | Oticon Ruby 1 | Oticon Ruby 2 |
|----------------------|--|---------------------|---------------------|
| Speech Understanding | Multiband Adaptive Directionality LX | • | • |
| | Single Compression LX | • | • |
| | Noise Reduction LX | • | • |
| | Speech Rescue™ LX | • | • |
| Sound Quality | Fitting Bandwidth | 8 kHz | 8 kHz |
| | Frequency Channels | 48 | 48 |
| Listening Comfort | Transient Noise Management | On/Off | |
| | SuperShield | • | |
| | Feedback shield LX | • | • |
| | Wind Noise Management | • | • |
| | Tinnitus SoundSupport™ | • | • |
| Fitting Options | Fitting Bands | 10 | 8 |
| | Fitting Formulas | NAL-NL1+2, DSL v5.0 | NAL-NL1+2, DSL v5.0 |
| | Oticon CROS/BiCROS compatible | • | • |
| Connectivity | Wireless streaming (2.4 GHz) | • | • |
| | Full wireless device compatibility | • | • |
| | Oticon ON App | • | • |
| | DAI/FM | • | • |
| Reliability | IP68 certified water and dust resistance | • | • |




Oticon Ruby BTE style colors

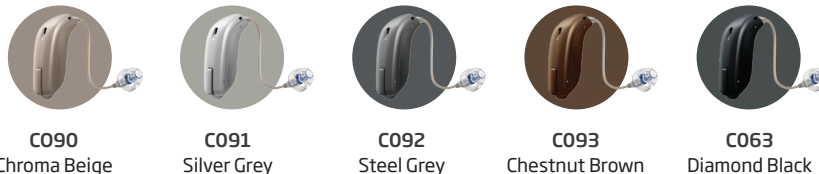


OTICON | Siya

Custom instruments at the essential level are available with Oticon Siya. Powered by the ultra-fast Velox™ platform, Siya custom instruments deliver excellent sound quality, advanced sound processing and 2.4 GHz wireless connectivity options. Offered in a variety of colors, styles and power levels, Siya custom helps your patients make the most of every moment.

| Benefits | Features | Oticon Siya 1 | Oticon Siya 2 |
|----------------------|--|---------------------|---------------------|
| Speech Understanding | Multiband Adaptive Directionality LX | ● | ● |
| | Single Compression LX | ● | ● |
| | Noise Reduction LX | ● | ● |
| | Speech Rescue™ LX | ● | ● |
| Sound Quality | Fitting Bandwidth | 8 kHz | 8 kHz |
| | Frequency Channels | 48 | 48 |
| Listening Comfort | Transient Noise Management | On/Off | |
| | Feedback shield LX | ● | ● |
| | Wind Noise Management | ● | ● |
| | Tinnitus SoundSupport™ | ● | ● |
| Fitting Options | Fitting Bands | 10 | 8 |
| | Fitting Formulas | NAL-NL1+2, DSL v5.0 | NAL-NL1+2, DSL v5.0 |
| Connectivity | Wireless streaming* (2.4 GHz) | ● | ● |
| | Full wireless device compatibility | ● | ● |
| | Oticon ON App | ● | ● |
| Reliability | IP68 certified water and dust resistance | ● | ● |

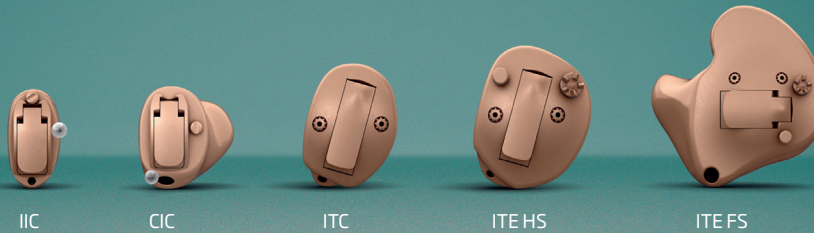

Oticon Siya Custom style colors

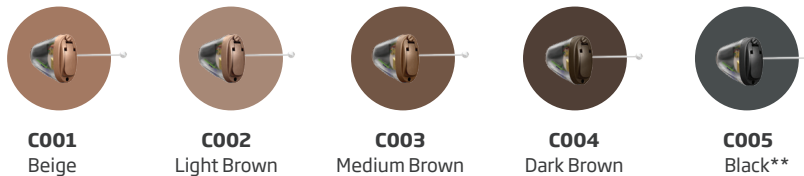

*For styles with 2.4 GHz
**Oticon Siya IIC only

For more information about Oticon Ruby and Oticon Siya contact
your Oticon representative or go to:

www.oticon.com/oticonruby



**Absolute
Hearing Solutions®**

Absolute Hearing Solutions LLC
absolutehearing@att.net

750 Cross Pointe Road Suite F
Gahanna, Ohio 43230

Phone: 614-452-4280
Toll Free: 888-803-2159
Fax: 614-577-0481
www.absolutehearingsolutions.com



www.oticon.com

Oticon is part of the Demant Group.

oticon
life-changing technology