

Phonak Fast Facts



Phonak RemoteMic

What is it?

Phonak RemoteMic is a lightweight (13g) wireless microphone, worn by the conversation partner whose speech is streamed via Bluetooth to the Phonak ComPilot, ComPilot II or ComPilot Air II and relayed to the hearing instrument(s) wirelessly.

Why use it?


The benefit for users is that they can continue a 1:1 conversation over distance even if they are not facing the speaker e.g. on a golf course. They simply hand over the RemoteMic to the speaker, who switches it on and attaches it to their lapel.

How is it used?

The RemoteMic is available as a stand-alone solution ready to be combined with an existing ComPilot, ComPilot II or ComPilot Air II. It is also available as a complete set (Conversation Set or ComPilot/RemoteMic Set) The devices are "ready-to-go" but can also be optimized to the individual hearing loss in the Phonak Target™ fitting software.



Fitting > Streaming programs > RemoteMic / Roger + mic

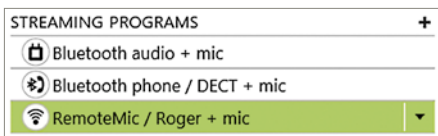

For more detailed information visit:
www.phonakpro.com/remotemic



Absolute Hearing Solutions LLC
absolutehearing@att.net

750 Cross Pointe Road Suite F
Gahanna, Ohio 43230

Phone: 614-452-4280
Toll Free: 888-803-2159
Fax: 614-577-0481

www.absolutehearingsolutions.com

PHONAK
life is on